

INDICAID™

Developed in
Hong Kong

- ✓ Flu season
- ✓ Immunocompromised people
- ✓ Exposed in high-risk areas



COVID-19 / FLU A&B RAPID ANTIGEN TEST



Detect 3 viruses at once
(Coronavirus, Flu A&B viruses)



Suitable for 2+ years old*



Results in 15 mins



For symptomatic &
asymptomatic individuals



Painless shallow
nasal swab



99% Accuracy*


* Swabbing should be performed by an adult for children aged 2 to 13. Users must be aged 14 or above to perform self-test.
* More information on website.

Similar Symptoms with Different Treatments


Flu A & B	Mild Symptoms Take adequate rest and drink plenty of water	>	Serious / Persistent Symptoms Seek medical advice promptly (Take doctor-prescribed antibiotics if complicated by bacterial infection)	High-Risk Groups for Flu Complications (incl. children, ages 50+ and chronically ill patients) Take antiviral medicines prescribed by doctors in the first 48 hours of flu-like symptoms.
	No/Mild Symptoms Home quarantine or enter isolation facilities	>	Serious Symptoms Hospitalized for treatments	

Instruction

How-to-use
Video

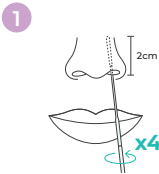


Gently insert the swab **2cm into each of your nostril**, rub the swab against the wall of your nostrils at least **4 times** in a large circular path.

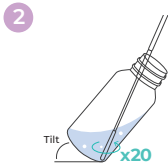


Stir the swab into the buffer solution **20 times**.

Tilt the vial to ensure the swab tip is fully submerged in the solution.

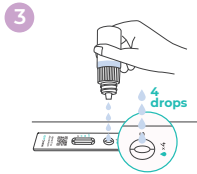

Drip **4 drops** of the solution into the oval opening of the test device.


Place the test device on a **flat surface** until the test is completed.

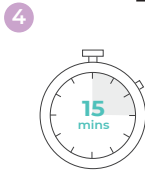

Wait 15 minutes for the result.

NOT to read the results after 20 mins; otherwise, the result will be considered as invalid.

Result Interpretations

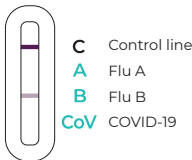

INVALID No 'C' Line	NEGATIVE 'C' Line Only	POSITIVE 'C' Line AND any 'A', 'B', or 'CoV' Line
 If no 'C' line is seen, the test result is INVALID even if you see 'A', 'B', or 'CoV' line(s).		 Look very closely! Even a very faint line next to the 'A', 'B', or 'CoV' should be considered positive if there is also a 'C' line.

Specifications



Individually-wrapped
Test Device



User Instructions



Buffer
Solution Vial



Individually-wrapped Swab

Product Code & Packaging	P0098 (1kit) P0099 (12kits)	Accuracy	COVID-19: Sensitivity 94.7%, Specificity 100% Flu A/B: Sensitivity 85.7%, Specificity 100% *Prospective clinical study pending for Flu B
NDC Numbers	4 897116 730459 (1-kit) 4 897116 730503 (12-kit)	Storage Condition	2°-30°C / Do not freeze / Avoid direct sunlight
Testing Method	Self-test	Shelf Life	6 months
Specimen Type	Shallow nasal swab		
Intended Use	INDICAID™ COVID-19/FLU A&B RAPID ANTIGEN TEST is an in vitro diagnostic test for determining the presence of SARS-CoV-2, influenza A (flu A) and/or influenza B (flu B) antigens in anterior nasal swab samples		
Components	1-kit: 1x individually-wrapped test device, buffer solution vial, individually-wrapped swab, user instructions 12-kit: 12x individually-wrapped test devices, buffer solution vials, individually-wrapped swabs; 1x user instructions		

